

BAT Mock Test Breakdown

The mock test consists of the following 7 breath test situations. The situations are gone over in detail below.

Items you require to complete this mock:

- 7 DOT ATFs (you may use a photocopy for the mock as we will only need one copy)
- Listerine/Scope/Mouthwash spray WITH alcohol (to use for donor to rinse with to “blow over” for the mock)
- Approved Evidential Breath Testing device
- 10 breathalyzer mouthpiece
- Tamper evident tape
- Dry gas/wet bath for breathalyzer accuracy check
- Regulator for breathalyzer accuracy check

Situation 1

POS-POS - This will be a positive screen, with a positive confirmation (after the required 15-minute wait). **You WILL need to complete an accuracy check** after the positive confirmation, just as you would under normal testing circumstances. Make sure to have Listerine or Scope (with alcohol in it!) to use for the blow over, as well as your dry gas and regulator to complete your accuracy check.

Situation 2

POS-NEG - This will be a positive screening test followed by a negative confirmation test (after the required 15-minute wait).

Situation 3

POS-Refusal - This will be a positive screening test followed by the donor refusing to comply with the confirmation test.

Situation 4

This will be a situation where the donor is unable to produce enough breath volume to meet the EBT's minimum volume requirements for an automatic test. Your mock “donor” will have to start blowing into the machine and then stop to fail the automatic test twice in a row. You will then have to go through your device procedures, explain what is being done to your donor, and perform a Manual Test.

Situation 5-7

These will be regular Negative screening situations.

For the mock testing, you will need to follow the appropriate procedures, treating each test as a new test with a new person (even though you are using the same mock “donor” for each test), and completing each ATF perfectly for each specific situation.

If you should have any questions about any of these situations go back through the modules to find the answers and make some notes to take into the mock test with you. Remember, the mock test is an OPEN BOOK TEST! You are allowed to have a procedural guide beside you or any of your notes to help you stay on track.

Please make sure to go over the ‘ATF’ module to make sure you know the proper way to fill out your ATFs!